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STERILE - SINGLE USE

1 - DESCRIPTION

The single use gynaecological forceps are designed for several interventions in the gynaecology field.

2 - REFERENCES AND CHARACTERISTICS

Reference	Product	Raw material	Length
PS3180025	Straight Pozzi forceps	Polyamid	25 cm
PS3210025	Curved Cheron forceps	ABS	25 cm
PS3580022	IUD string distal scissors	ABS and stainless steel	22 cm
PS4190000	IUD insertion kit composed of: - Straight Pozzi forceps - Curved Cheron forceps: to grasp the compress and disinfect the cervix - IUD string distal scissors (PS3580022)		

Sterilization method: sterilization by ethylene oxide.

3 - INDICATIONS

The Pozzi forceps are used for the prehension of the cervix to maintain the cervix during gynaecological interventions.

The Cheron forceps are used for the cervix cleaning and care (bandage, compress...). It is also used with IUD insertion kit to hold the thread of the coil.

IUD string scissors are used to cut the thread of the coil.

4 - RECOMMENDATIONS

- Read the instructions carefully before using.
- This medical device must only be used by trained and qualified medical staff or authorised person, familiar with gynaecological techniques and acquainted with the pathologies in question as well as their possible complications.

DO NOT USE the device if the packaging has been opened or damaged, or if it shows any defects caused by transport, incorrect storage conditions, or incorrect handling which might adversely affect its use.

5 - STORAGE CONDITIONS

In order to prevent any damage, products should be stored in their original packaging, in a cold and dry place.

6 - PREPARATION OF MATERIALS - USE

- The medical devices are delivered STERILE and for SINGLE USE.
- Make sure the packaging is intact and the shelf-life has not expired as these aspects guarantee the sterility and quality of the product.
- Open the sachet by peeling off the paper.
- Remove the medical devices according to the asepsis standards.
- Insert the medical devices into the speculum cavity to make the needed intervention.

7 - DISPOSAL OF THE PRODUCT AFTER USE

After using, dispose of the product and its packaging according to procedures in force at your establishment for the disposal of medical waste.

We remind you that these products are strictly disposable devices which cannot be reused.

In case of reuse, their performances could be affected with an important risk of contamination.

SINGLE USE - DO NOT RE-USE

8 - SIGNIFICATION OF SYMBOLS USED ON LABELS

	LOT	REF	STERILE EO							
Expiry date	Lot number	Product reference	Ethylene oxide sterilisation	Do not use if packaging is damaged	Do not reuse	Read the instructions for use	Avoid heat	Avoid humidity	Manufacturing date	Manufacturer

Manufacturer :

Prince Medical - 64, rue Gutenberg - ZA La Sente du Moulin - 60530 Ercuis
FRANCE

Tel. : + 33 (0)3 44 26 54 76 - Fax : + 33 (0)3 44 26 54 95